

## INSTRUCTIONS FOR USE — EYE

1




**In Pre-Op:** Confirm the correct eye with the patient. Remove clear plastic covering on tattoo. The writing on the front of the tattoo is reversed at this point. After it is turned face down on the skin it will read correctly. Press color side of the tattoo onto the skin. It is a little sticky and will stay in place. Cover the tattoo with saline wipe – no need to unfold wipe.

2



**Mark PROVIDER Box:** Have a second provider confirm that the tattoo has been placed above the correct eye to be operated on and mark the box labelled PROVIDER using the enclosed marking pen.

3

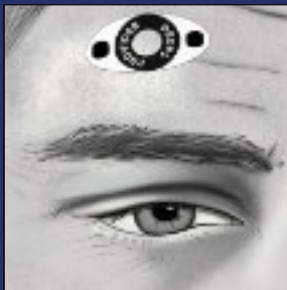

**Mark the DROPS Box:** when applying a topical anesthetic to the eye, confirm that the tattoo has been placed above the correct operative eye and then mark the box labelled DROPS after administering the medication.

4



**Final confirmation:** Prior to starting the procedure, confirm the correct operative eye has been identified and anesthetized by locating the tattoo with the PROVIDER and DROPS boxes marked. When ready, the tattoo can be wiped away with an alcohol wipe or moistened gauze.